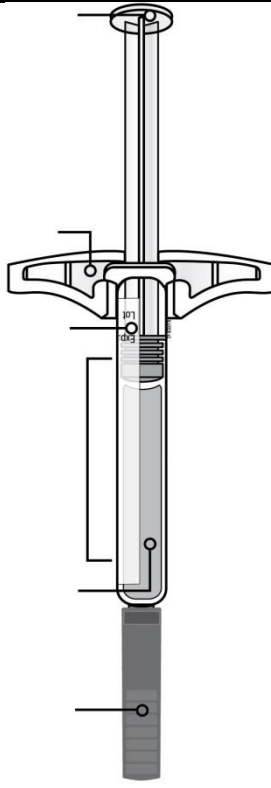
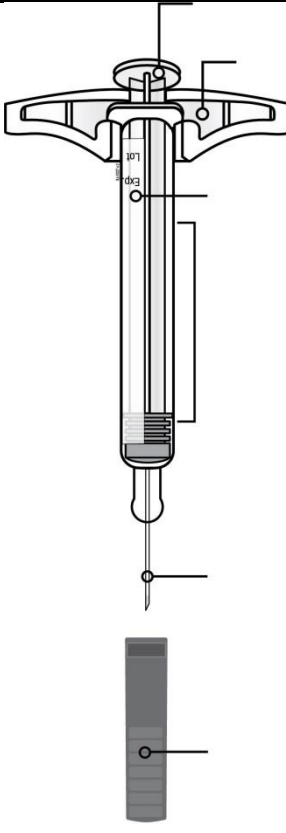


Instructions for use:
AMGEVITA® single use prefilled syringe
For subcutaneous use

Guide to parts

Guide to parts	
Before use	After use
<p>Plunger rod</p> <p>Finger flange</p> <p>Label and expiration date</p> <p>Syringe barrel</p> <p>Medicine</p> <p>Needle cap on</p> 	<p>Used plunger rod</p> <p>Finger flange</p> <p>Label and expiration date</p> <p>Used syringe barrel</p> <p>Used needle</p> <p>Needle cap off</p> 
Important: Needle is inside	

Important

Before you use an AMGEVITA prefilled syringe, read this important information:



Using your AMGEVITA prefilled syringe

- It is important that you do not try to give the injection unless you or your caregiver has received training.
- **Do not** use an AMGEVITA prefilled syringe if it has been dropped on a hard surface. Part of the AMGEVITA prefilled syringe may be broken even if you cannot see the break. Use a new AMGEVITA prefilled syringe.

Step 1: Prepare

- A.** Remove the number of AMGEVITA prefilled syringes you need from the package.

Grab the syringe barrel to remove the syringe from the tray.

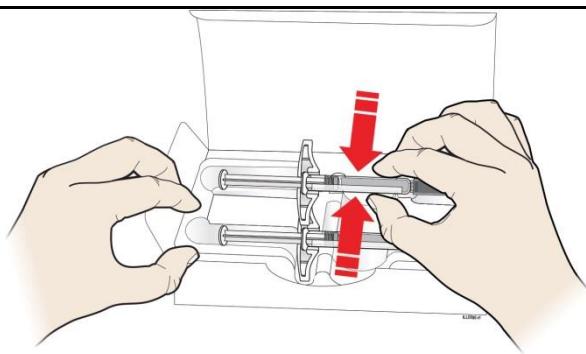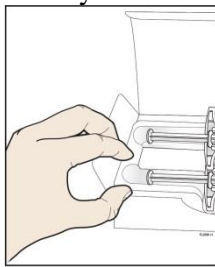

Grab Here

Place your finger or thumb on edge of tray to secure it while you remove the syringe.

Put the original package with any unused syringes back in the refrigerator.

For safety reasons:


- **Do not** grasp the plunger rod.
- **Do not** grasp the needle cap.
- **Do not** remove the needle cap until you are ready to inject.
- **Do not** remove the finger flange. This is part of the syringe.

For a more comfortable injection, leave the syringe at room temperature for **15 to 30** minutes before injecting.

- **Do not** put the syringe back in the refrigerator once it has reached room temperature.
- **Do not** try to warm the syringe by using a heat source such as hot water or microwave.
- **Do not** leave the syringe in direct sunlight.
- **Do not** shake the syringe.

Important: Always hold the prefilled syringe by the syringe barrel.

B. Inspect the AMGEVITA prefilled syringe.

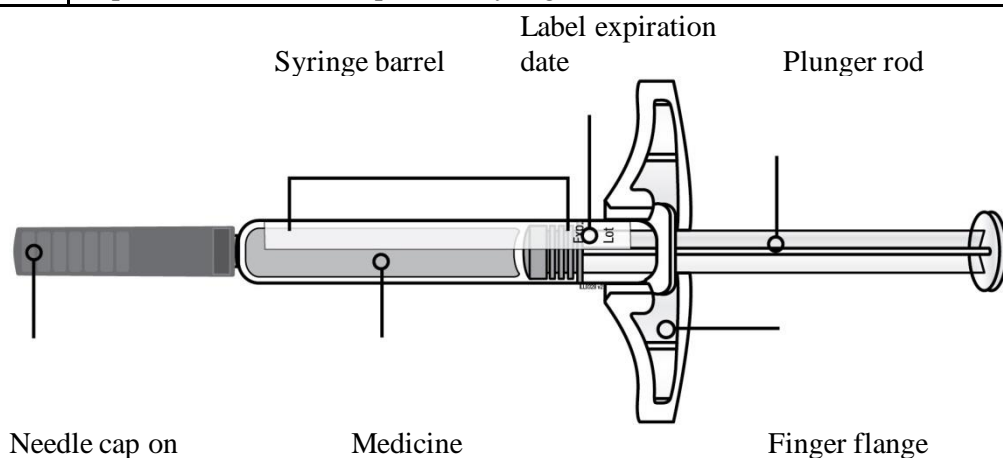

Always hold the syringe by the syringe barrel.

Make sure the medicine in the syringe is clear and colourless to slightly yellow.

- **Do not** use the syringe if:
 - The medicine is cloudy or discoloured or contains flakes, or particles.
 - Any part appears cracked or broken.
 - The needle cap is missing or not securely attached.
 - The expiration date printed on the label has passed.

In all cases, use a new syringe.


C. Gather all materials needed for your injection(s).

Wash your hands thoroughly with soap and water.


On a clean, well-lit work surface, place a new, prefilled syringe.

You will also need these additional items, as they are not included in the carton:

- Alcohol wipes
- Cotton ball or gauze pad
- Plaster
- Sharps disposal container

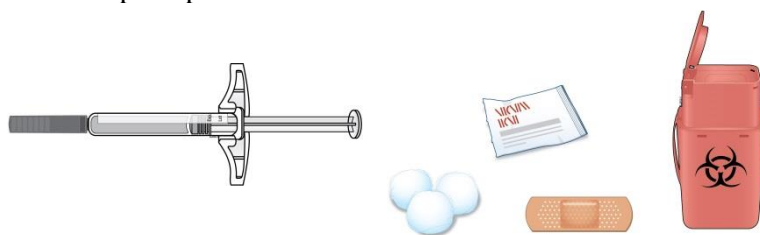

D. Prepare and clean your injection site(s).

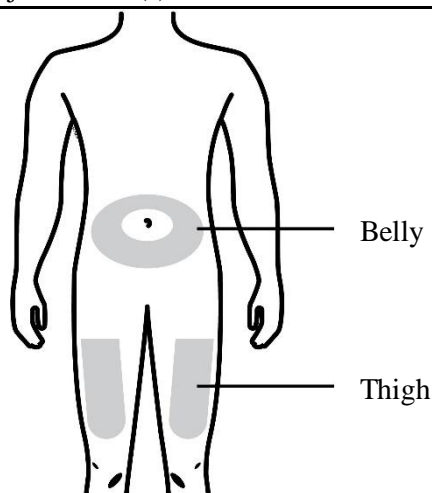

You can use:


- Your thigh
- Belly, except for a 2 inch (5 centimetres) area around your belly button

Clean your injection site with an alcohol wipe. Let your skin dry.

- **Do not** touch this area again before injecting.
- If you want to use the same injection site, make sure it is not the same spot on the injection site you used for a previous injection.
 - **Do not** inject into areas where the skin is tender, bruised, red, or hard. Avoid injecting into areas with scars or stretch marks.
- If you have psoriasis, you should avoid injecting directly into raised, thick, red, or scaly skin patch or lesion.

Step 2: Get ready

E. Pull the needle cap straight out and away from your body when you are ready to inject.

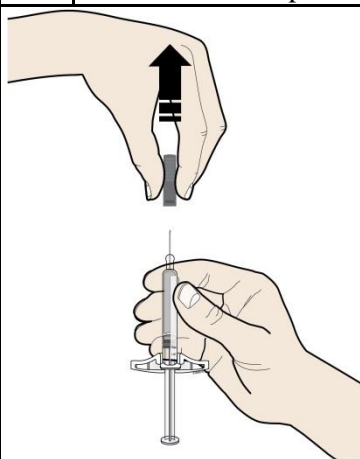


It is normal to see a drop of liquid at the end of the needle.

- **Do not** twist or bend the needle cap.
- **Do not** put the needle cap back onto the syringe.
- **Do not** remove the needle cap from the syringe until you are ready to inject.

Important: Throw the needle cap into the sharps disposal container provided.

F. Pinch your injection site to create a firm surface.

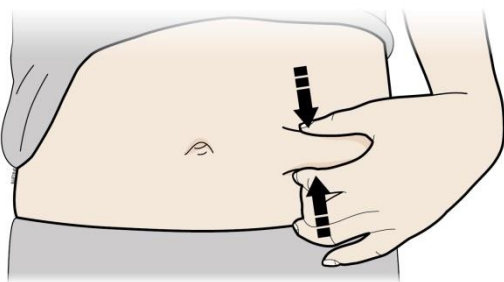


Pinch the skin firmly between your thumb and fingers, creating an area about 2 inch (5 centimetres) wide.

Important: Keep the skin pinched while injecting.


Step 3: Inject

G. Hold the pinch. With the needle cap off, insert the syringe into your skin at 45 to 90 degrees.

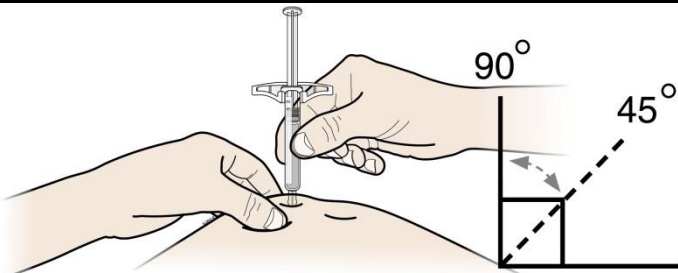


Do not place your finger on the plunger rod while inserting the needle.

H. Using slow and constant pressure, push the plunger rod all the way down until it stops moving.

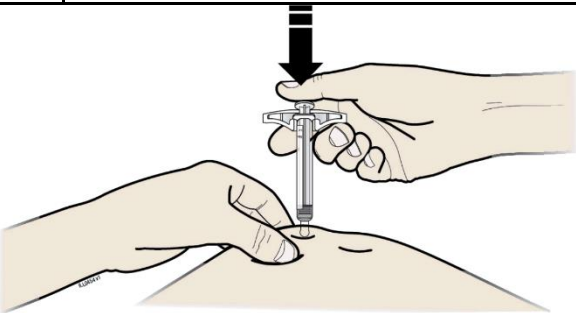

I. When done, release your thumb, and gently lift the syringe off of your skin.

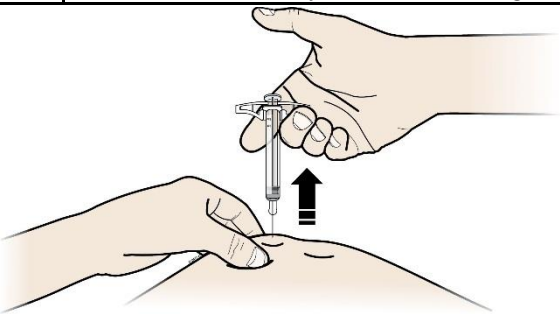

Step 4: Finish

J. Discard the used syringe and the needle cap.



- **Do not** reuse the used syringe.
- **Do not** use any medicine that is left in the used syringe.
- Put the used AMGEVITA syringe in a sharps disposal container immediately after use. **Do not** throw away (dispose of) the syringe in your household waste.
- Talk with your doctor or pharmacist about proper disposal. There may be local guidelines for disposal.
- **Do not** recycle the syringe or sharps disposal container or throw them into the household waste.

Important: Always keep the sharps disposal container out of the sight and reach of children.

K. Examine the injection site.

If there is blood, press a cotton ball or gauze pad on your injection site. **Do not** rub the injection site. Apply a plaster if needed.

December 2020

SGAMGPFSIFU03

AMGEN[®]

AMGEVITA[®] is a registered trademark owned or licensed by Amgen Inc., its subsidiaries, or affiliates.