

Important information you need to know before injecting MOUNJARO

Read this Instructions for Use and the Package Insert before using your MOUNJARO Pen and each time you get a refill. There may be new information. This information does not take the place of talking to your healthcare provider about your medical condition or treatment.


Talk to your healthcare provider about how to inject MOUNJARO the right way.

- MOUNJARO is a single-dose prefilled pen.
- MOUNJARO is used 1 time each week.
- Inject under the skin (subcutaneously) only.
- You or another person can inject into your stomach (abdomen) or thigh.
- Another person can inject into the back of your upper arm.

Storage and handling

- Store your Pen in the refrigerator between 2°C to 8°C (36°F to 46°F).
- You may store your Pen at room temperature up to 30°C (86°F) for up to 21 days.
- **Do not** freeze your Pen. If the Pen has been frozen, throw the Pen away and use a new Pen.
- Store your Pen in the original carton to protect your Pen from light.
- The Pen has glass parts. Handle it carefully. If you drop the Pen on a hard surface, **do not** use it. Use a new Pen for your injection.
- Keep your MOUNJARO Pen and all medicines out of the reach of children.

Guide to parts

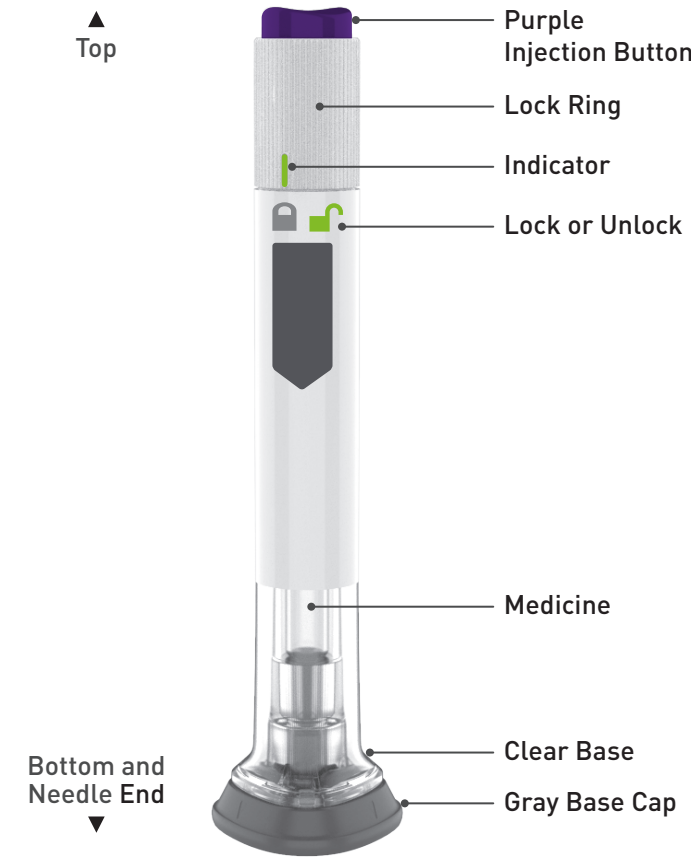


Preparing to inject MOUNJARO

Remove the Pen from the refrigerator.

Leave the gray base cap on until you are ready to inject.

Check the Pen label to make sure you have the right medicine and dose, and that it has not expired.

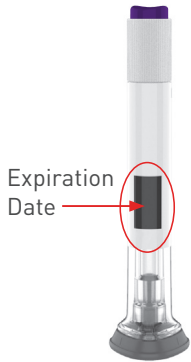


Inspect the Pen to make sure that it is not damaged.

Make sure the medicine:

- is not frozen
- is colorless to slightly yellow
- is not cloudy
- does not have particles


Wash your hands.

Step 1 Choose your injection site

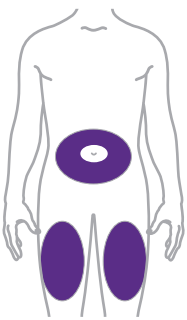

Your healthcare provider can help you choose the injection site that is best for you.

You or another person can inject the medicine in your stomach (abdomen) or thigh.

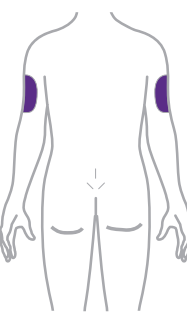


Another person should give you the injection in the back of your upper arm.

Change (rotate) your injection site each week.

You may use the same area of your body but be sure to choose a different injection site in that area.

Step 2 Pull off the gray base cap

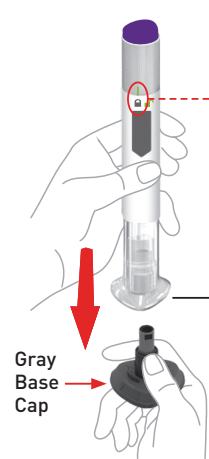

Make sure the Pen is **locked**.
Do not unlock the Pen until you place the clear base on your skin and are ready to inject.


Clear Base

Pull the gray base cap straight off and throw it away in your household trash.


Do not put the gray base cap back on – this could damage the needle.

Do not touch the needle.

Step 3 Place clear base on skin, then unlock

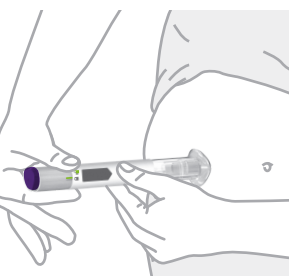


Place the clear base flat against your skin at the injection site.

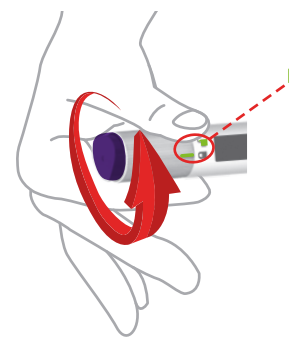

Unlock by turning the lock ring.

Step 4 Press and hold up to 10 seconds

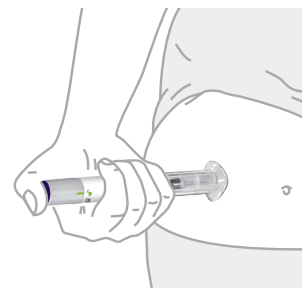

Press and hold the purple injection button for up to 10 seconds.

Listen for:

- First click = injection started
- Second click = injection completed



You will know your injection is complete when the gray plunger is visible.

After your injection, place the used Pen in a sharps container.
See **Disposing of your used Pen.**

PPD Information Box	
Technical Information:	
<div><div>BLACK</div><div>368</div><div>485</div><div>669</div><div>2597</div><div>Cool Gray 11</div></div>	<div><div>DIE CUT</div></div>
Previous Item Code (to be destroyed)	N/A
Lilly	

- Put your used Pen in a cleared sharps disposal container right away after use. **Do not** throw away (dispose of) Pens in your household trash.
- If you do not have a cleared sharps disposal container, you may use a household container that is:
 - made of a heavy-duty plastic,
 - can be closed with a tight-fitting, puncture-resistant lid, without sharps being able to come out,
 - upright and stable during use,
 - leak-resistant, and
 - properly labeled to warn of hazardous waste inside the container.
- When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container.
- **Do not** recycle your used sharps disposal container.

Lilly