





10.05.2017 09:10:33

N19B1404A.indd 28

GONAL- PRE-FILLED PEN 900 IU/1.5 mL

GONAL-0 900 IU/1.5 ml (66 micrograms/1.5 ml)

Instructions for use



N19B1404A.indd 1

10.05.2017 09:10:33

Contents

- 1. How to use the GONAL-f pre-filled pen
- 2. How to use your GONAL-f pre-filled pen Treatment diary
- 3. Before you start using your GONAL-f pre-filled pen
- 4. Getting your GONAL-f pre-filled pen ready for injection
- 5. Setting the dose prescribed by your doctor
- 6. Injecting the dose
- 7. After the injection
- 8. GONAL-f pre-filled pen Treatment diary (see table at the end)

Warning: Please read these instructions for use before using your GONAL-f pre-filled pen. Follow the procedure exactly, as it may differ from your past experience.

1. How to use the GONAL-f pre-filled pen

- Do not share the pen. The pen is for subcutaneous injection only.
- The numbers on the Dose Feedback Window are measured in International Units or IUs. Your doctor will
 have told you how many IUs to inject each day.
- The numbers displayed in the Dose Feedback Window help you to:

a. Dial your prescribed dose.

b. Verify a complete injection.

c. Read the dose remaining to be injected with a second pen.

- Give yourself the injection at the same time each day.
- Your doctor/pharmacist will tell you how many pens you need to complete your treatment.



- 3 -







Example 7:00 am

2. How to use your GONAL-f pre-filled pen Treatment Diary

A treatment diary is included on the last page.

Use the treatment diary to record the amount of injected IUs you use each time.

- Record the treatment day number (1), date (2) and time (3) of your injection.
- In the first line of the table the volume of your pen is already recorded for you (4).
- Record your prescribed dose in the "Prescribed Dose" section (5).
- Verify you dialed the right dose before injecting (6).
- After injection, read the number shown in the Dose Feedback Window.
- Confirm you receive a complete injection (7) or record the number shown in the **Dose Feedback Window** if other than "0" (8).
- When needed, inject yourself using a second pen, dialing your remaining dose written in the "Amount Displayed After Injection" section (8).
- Record this remaining dose in the "Amount Set to Inject" section in the next row (6).

CAUTION:

Using your treatment diary to record your daily injection(s) allows you to verify every day that you received the full prescribed dose.

- 4 -

An example of a treatment diary:

1 Treatment Day	2 Date	3 Time	4 Pen Volume 900 IU/1.5 mL	5 Prescribed Dose	6	7 8 Dose Feedback Window	
Number					Amount Set to Inject	Amou	nt Displayed After Injection
#1	10 06	07:00	900 IU	350	350	✓ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
#2	11 06	07:00	900 IU	350	350	☑ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
#3	12 06	07:00	900 IU	350	350	□if "0", injection complete	If not "0", need second injection Inject this amount .150 using new pen
#3	12 06	07:00	900 IU	N/A	150	✓ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen

Note: The 300 IU pen maximum single-dose setting is 300 IU; the 450 IU pen maximum single-dose setting is 450 IU; the 900 IU pen maximum single-dose setting is 450 IU.

- 5 -

3. Before you start using your GONAL-f pre-filled pen

- Wash your hands with soap and water.
- Find a clean area and a flat surface.
- Verify the expiration date on the pen label.
- Gather everything you need and lay it out:









- 1. Dose setting knob
- 2. Dose Feedback Window
- 3. Plunger piston
- 4. Reservoir holder

- 5. Threaded needle connector
- 6. Pen cap
- 7. Peel-off seal tab
- 8. Removable needle

- 9. Inner needle shield
- 10. Outer needle cap
- 11. Alcohol swabs
- 12. Sharps disposal container

- 7 -

4. Getting your GONAL-f pre-filled pen ready for injection

4.1. Take off the pen cap

4.2. Verify that the Dose Feedback Window is set to "0".



4.3. Prepare your needle for injection

- Get a new needle only use the "single-use" needles supplied.
- Hold the outer needle cap firmly.
- Check that the peel-off seal on the outer needle cap is not damaged or loose:
- Remove the peel-off seal.







CAUTION:

If the peel-off seal is damaged or loose, do not use the needle. Throw it away in a sharps disposal container. Get a new needle.

-9-

4.4. Attach the needle

• Screw the threaded tip of the GONAL-f pre-filled pen into the outer needle cap until you feel a light resistance.

Warning: Do not attach the needle too tightly; the needle could be difficult to remove after the injection.







• Remove the outer needle cap by pulling it gently. Put it aside for later use.

- Hold the GONAL-f pre-filled pen with the needle pointing upward.
- Carefully remove and discard the green inner shield.



4.5. Look closely at the tip of the needle for tiny drop(s) of fluid

• If you see a tiny drop(s) of fluid proceed to Section 5: Setting the dose prescribed by your doctor.

Warning: ONLY check for drop(s) the FIRST TIME you use a new GONAL-f pre-filled pen to remove air in the system.



CAUTION:

If you do not see a tiny drop(s) at or near the needle tip **the first time** you use a new pen, you must perform the steps on the next page.

- 11 -





1. Gently turn the dose setting knob clockwise until it **reads 25** in the Dose Feedback Window. You can turn the dose knob backwards if you turn it past 25.





- 2. Hold the pen with the needle pointing upward.
- 3. Tap the reservoir holder gently.
- 4. Press the dose setting knob as far as it will go. A tiny drop of liquid will appear at the tip of the needle.
- 5. Verify that the Dose-Feedback Window reads "0".
- 6. Proceed to Section 5: Setting the dose prescribed by your doctor.

- 13 -

5. Setting the dose prescribed by your doctor

- **5.1.** The pen contains 900 IU follitropin alpha.
 - The 900 IU pen maximum single-dose setting is 450 IU and the smallest single-dose setting is 12.5 IU.

5.2. Turn the dose setting knob until your intended dose shows in the Dose Feedback Window.



• Turn the dose setting knob forward to dial up



• Turn the dose setting knob **backward** to correct the dose

- 14 -

5.3. Set the dose that was prescribed by your doctor (in the example shown in the figure, it is 50 IU).



Warning: Check that the Dose Feedback Window displays your complete prescribed dose before you move on to the next step.

- 15 -

6. Injecting the dose

- 6.1. Choose an injection site in the area your doctor or nurse has told you to give the injection. To minimize skin irritation, select a different injection site each day.
- 6.2. Clean the skin by wiping with an alcohol swab.
- **6.3.** Verify once more that the Dose Feedback Window displays the correct dose.
- 6.4. Inject the dose as you were trained to do by your doctor or nurse.
 - Slowly push the needle into the skin entirely (1).
 - Press the dose knob down as far as it will go and hold it to complete the full injection.





- Hold the dose knob down for a minimum of 5 seconds to ensure you inject the full dose (2). The larger the dose, the longer it will take to inject.
- The dose number shown in the Dose Feedback Window will turn back to 0.

- After a minimum of 5 seconds, pull the needle out of the skin while keeping the dose setting knob pressed down (3).
- Release the dose setting knob.
- Warning: Always make sure to use a new needle for each injection.



7. After the injection

7.1. Verify you have given a complete injection.

• Check that the Dose Feedback Window shows "0".



Warning: If the Dose Feedback Window shows a number higher than 0, the GONAL-f pre-filled pen is empty and you have not received your full prescribed dose.

- 18 -

7.2. Complete a partial injection (only when needed).

• The Dose Feedback Window will indicate the missing amount you need to inject using a new pen.



- Repeat Section 3 ("Before you start using your GONAL-f pre-filled pen") through Section 4 ("Getting your GONAL-f pre-filled pen ready for injection") with a second pen.
- Set the dose to the missing amount you recorded in the treatment diary or the number still displayed in the Dose Feedback Window on your previous pen, and inject

- 19 -

7.3. Removing the needle after each injection.

- Place the outer needle cap on a flat surface.
- Hold the GONAL-f pre-filled pen firmly with one hand, and slip the needle into the outer needle cap.
- Continue by pushing the capped needle against a firm surface until you hear a "click."





 Grip the outer needle cap and unscrew the needle by turning anti-clockwise. Dispose of the used needle safely.



- Never reuse any used needle. Never share needles.
- Recap the pen.



7.4. Storing the GONAL-f pre-filled pen.

CAUTION:

Never store the pen with the needle attached.

Always remove the needle from the GONAL-f pre-filled pen before replacing the pen cap.

- Store the pen in its original packaging in a safe place.
- When the pen is empty, ask your pharmacist how to dispose of it.

Warning: Medicine should not be disposed of via wastewater or household waste.

- 21 -

10.05.2017 09:11:00

8. GONAL-f pre-filled pen Treatment Diary

1 Treatment Day Number	2 Date	3 Time	4 Pen Volume 900 IU/1.5 mL	5 Prescribed Dose	6 7 8 Dose Feedback Window		
				Amount Set to Inject			
	1	:	900 IU			☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
	1	:	900 IU			☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
	1	:	900 IU			☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
	1	:	900 IU			☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
N19B1404A.ind	d 22 ′		000 111			□if "0",	if not "0", need second injection 10.05.2017 09:11:01

, '				injection complete	Inject this amountusing new pen
1	:	900 IU		□if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
1	:	900 IU		□if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
1	:	900 IU		☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
1	:	900 IU		☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
1	:	900 IU		☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
1	:	900 IU		□if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen

- 23 -

1 Treatment Day Number	2 Date	3 Time	4 Pen Volume 900 IV/1.5 mL	5 Prescribed Dose	6 7 8 Dose Feedback Window		
				Amount Set to Inject			
	1	:	900 IU			☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
	1	:	900 IU			☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
	1	:	900 IU			☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
	1	:	900 IU			☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
N19B1404A.inc	ld 24 '		000 111			∏if "0",	□ if not "0", need second injection 10.05.2017 09:11:01

· ·		00010		injection complete	Inject this amountusing new pen
1	:	900 IU		□if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
1	:	900 IU		☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
1	:	900 IU		☐ if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
/	:	900 IU		□if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen
/	:	900 IU		□if "0", injection complete	☐ if not "0", need second injection Inject this amountusing new pen

Note: The 300 IU pen maximum single-dose setting is 300 IU; the 450 IU pen maximum single-dose setting is 450 IU; the 900 IU pen maximum single-dose setting is 450 IU. This Instructions for use was last revised in December 2015

- 25 -

N19B1404A.indd 26

10.05.2017 09:11:01

N19B1404A.indd 27

10.05.2017 09:11:01