Instructions for Use

ENTYVIO® (en ti' vee oh) Pen

(vedolizumab)

Injection, for subcutaneous use Single-dose prefilled pen/autoinjector

Important Information

Read and follow these Instructions for Use before you inject. Your healthcare provider should show you how to use the ENTYVIO prefilled pen/autoinjector before you use it for the first time.


- · Keep out of reach of children.
- Do not put or press thumb, fingers or hand over the yellow needle shield.

If you have any questions or concerns about your ENTYVIO prefilled pen/autoinjector, please call your healthcare provider.

Storage

- Refrigerate at 2°C to 8°C.
- If needed, for example when you are traveling, the prefilled pen/autoinjector can be left in its box at room temperature up to 25°C for up to 7 days.
- **Do not** freeze.
- Do not leave in direct sunlight.

Your ENTYVIO single-dose prefilled pen/autoinjector

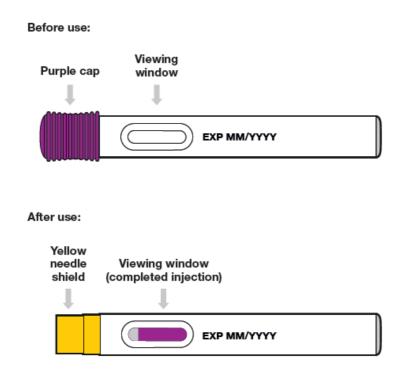

1) Gather supplies


- Find a clean, flat surface for your supplies.
- Take one (1) prefilled pen/autoinjector box from the refrigerator.
 - **Do not** use the prefilled pen/autoinjector if any of the seals on the box are broken or missing.
 - Check the expiration date on the box. Do not use if the expiration date on the box has passed.
- Wait 30 minutes to let the prefilled pen/autoinjector come to room temperature.
 - Do not warm the prefilled pen/autoinjector in any other way.
 - **Do not** let it sit in direct sunlight.
 - Do not take the prefilled pen/autoinjector out of its tray until you are ready to inject.
- Gather supplies that are not in the box.
 - Alcohol pad
 - Cotton ball or gauze
 - Sharps disposal container





2) Inspect the prefilled pen/autoinjector

- Wash your hands.
- Peel back the paper on the tray and lift the prefilled pen/autoinjector out.
- Inspect the prefilled pen/autoinjector for damage.
 - Do not use the prefilled pen/autoinjector if any part of it is damaged.
- Check the expiration date on the prefilled pen/autoinjector.
 - Do not use if the expiration date on the prefilled pen/autoinjector has passed.
- Check the medicine in the viewing window. It should be colorless to yellow.
 - **Do not** use the prefilled pen/autoinjector if the medicine is cloudy or has particles floating in it.
- You may see air bubbles in the prefilled pen/autoinjector.
 This is normal.
 - Do not shake.

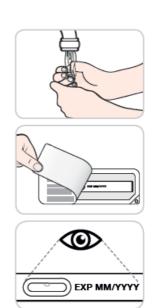


3) Prepare for the injection


- Choose an injection site on your bare skin from one of the following:
 - Front of the thighs, or
 - Stomach area (abdomen) except for the area 2 inches (5 cm) around the belly button (navel), or
 - Back of the upper arm (only if a caregiver gives the injection).
- Use a new injection site for each injection. If you want to use the same injection site, make sure it is not the same spot you used for the last injection.
 - **Do not** inject into moles, scars, bruises, or skin that is tender, hard, red, or damaged.
- Wipe the chosen site with an alcohol pad. Let your skin dry.
 - Do not touch this area again before you inject.
- Pull the purple cap straight off and throw it away.
 - **Do not** put or press thumb, fingers or hand over the yellow needle shield.
 - **Do not** re-cap the prefilled pen/autoinjector.
 - **Do not** use a dropped prefilled pen/autoinjector.


4) Inject ENTYVIO

- Hold the prefilled pen/autoinjector so you can see the viewing window.
- Place the prefilled pen/autoinjector at 90 degrees to the injection site.
 - Be sure the **yellow end is toward the injection site.**
 - Do not push down until you are ready to inject.
- Push down on the prefilled pen/autoinjector as far as it will go to begin the injection.
- Hold and count to 10 while pushing down with constant pressure. This will allow all of the medicine to be injected.
 - You may hear two clicks, one at the start and one near the end of the injection.
- Confirm that the viewing window is filled with purple before you stop pushing.
 - You will see a small amount of gray in the window. This is normal.

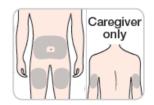



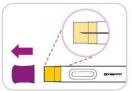


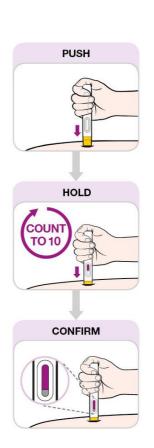

- Lift the prefilled pen/autoinjector from the injection site.
 - The yellow needle shield will drop down and lock over the needle.
 - If the viewing window did not fill completely, call your healthcare provider. You may not have received your full dose of medicine.
- You may see a small amount of blood at the injection site.
 If you do, press on your skin with a cotton ball or gauze.

5) Throw away supplies

- Put the used prefilled pen/autoinjector in a disposal container right away after use.
- Throw away the rest of the supplies, including the cap.
 - Do not recycle or throw away (dispose of) the prefilled pen/autoinjector in your household trash.

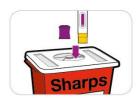

Version: Singapore Entyvio Pen IFU_October 2022