

Teriparatide-Teva
solution for injection in pre-filled pen
20 micrograms/80 microlitres

PHARMA CODE

CODE

PHARMA CODE

FOLDING LINE




Before you use your new Teriparatide-Teva Pen, please read the entire front and back of this User Manual completely. Follow the directions carefully when using the Teriparatide-Teva Pen. Also read the *Package Insert* that comes with your Teriparatide-Teva carton.

The Teriparatide-Teva Pen contains 28 days of medicine. Throw away the Teriparatide-Teva Pen after 28 days, even if it is not completely empty. Do not inject more than one dose of Teriparatide-Teva in the same day.

Do not transfer Teriparatide-Teva to a syringe.


Teriparatide-Teva Pen Parts*

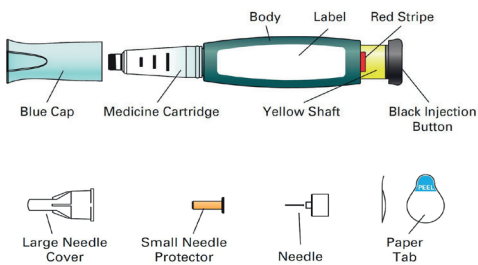

***Needles not included.** Pen needles from 29 to 31 gauge (Becton, Dickinson and Company) are recommended for use with this device. Ask your doctor or pharmacist which needle gauge and length are best for you. **Always use a NEW needle for each injection.**


Wash your hands before every injection. Prepare the injection site as your doctor or pharmacist instructed.

1. Pull off blue cap

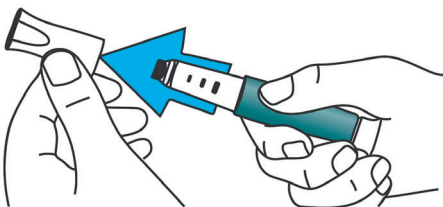


Check the Teriparatide-Teva Pen label to make sure you have the right medicine and that it has not expired. Do not use it if the Teriparatide-Teva Pen looks damaged, the medicine in the cartridge is not clear and colourless, or if it has particles in it.


2. Attach new needle

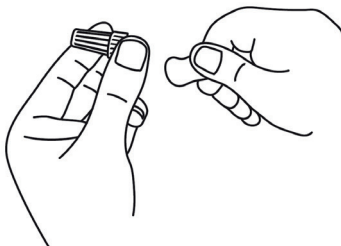


Pull off paper tab.

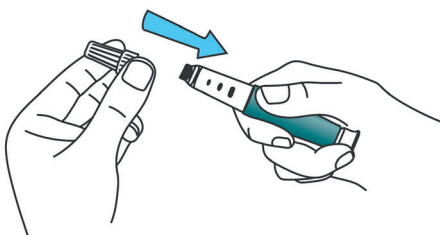

Push needle **straight** onto medicine cartridge.

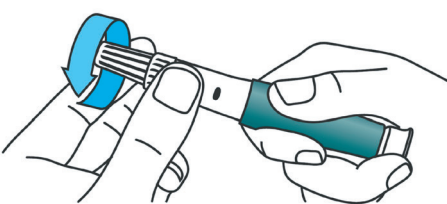


Screw on needle clockwise until it stops (3 to 5 clockwise turns). Pay attention that the needle is properly fixed.

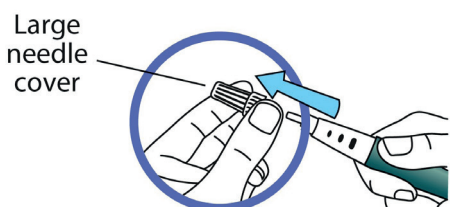


Pull off large needle cover and **save it**.


3. Set dose

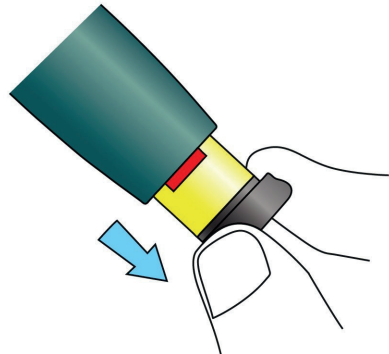

Pull out black injection button until **it stops** and the red stripe shows. If you cannot pull out the black injection button, see Troubleshooting, *Problem E*, on back page.

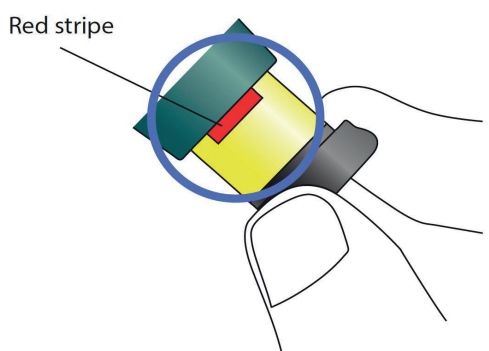


Check to make sure red stripe shows.

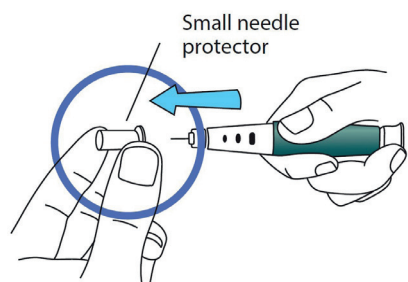

Pull off small needle protector and throw away.

4. Inject dose

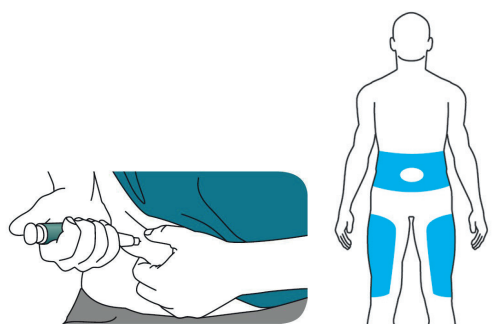

Gently hold a fold of skin on your thigh or abdomen and insert needle straight into skin.



Push in the black injection button **until it stops**. Hold the button and **count to 5 slowly**. You must wait until the count of 5 to make sure you receive the correct dose. Then pull the needle from your skin.

5. Confirm dose

IMPORTANT




After completing the injection:

Once the needle is removed from your skin, take your thumb off the black injection button. **Check** to make sure the black injection button is all the way in. If the yellow shaft does not show, you have finished the injection steps the right way.




You should **NOT** see any of the yellow shaft. If you do and have already injected the medicine, do not inject yourself a second time on the same day. Instead, you **MUST** reset the Teriparatide-Teva Pen (see Troubleshooting, Problem A, on back page).

6. Remove needle

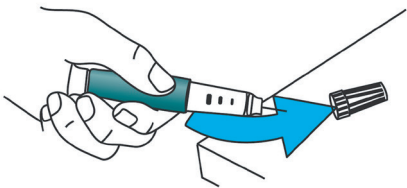


Put large needle cover on needle. Do not try to put the needle cover back on with your hands.

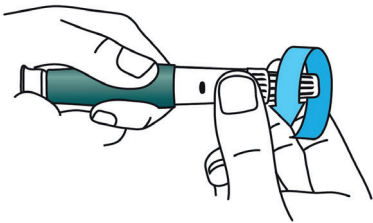

Unscrew the covered needle all the way by giving the large needle cover 3 to 5 counter-clockwise turns.



Pull off needle and throw away as directed by your doctor or pharmacist.

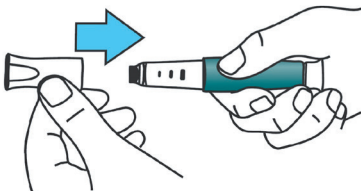

Push blue cap back on. Right after use, place Teriparatide-Teva Pen in the refrigerator.

For more information, or if you have any questions, turn to the <end><back> of this page.
Teriparatide-Teva injection in pre-filled pen



Troubleshooting

Problem	Solution
A. The yellow shaft is still showing after I push in the black injection button. How do I reset my Teriparatide-Teva Pen?	<p>You can prevent this problem by always using a NEW needle for each injection, ensuring the needle is properly attached, and by pushing the black injection button all the way in.</p> <p>To reset the Teriparatide-Teva Pen, follow the steps below.</p> <ol style="list-style-type: none">1) If you have already injected, DO NOT inject yourself a second time on the same day.2) Remove the needle.3) Repeat step 2 'Attach new needle' and step 3 'Set dose' of the instruction.4) Point the needle down into an empty container. Push in the black injection button until it stops. Hold it in. You may see a small stream or drop of fluid. When you have finished, the black injection button should be all the way in. <p>Continue with step 6 'Remove needle'.</p> <ol style="list-style-type: none">5) If you still see the yellow shaft showing, contact your doctor or pharmacist. <p>Repeated resetting of your Teriparatide-Teva Pen may lead to loss of daily doses.</p>



B. How can I tell if my Teriparatide-Teva Pen works?	<p>The black injection button should be all the way in to show that the full dose of medicine has been injected from the Teriparatide-Teva Pen.</p> <p>Use a new needle every time you inject to be sure your Teriparatide-Teva Pen will work properly.</p>
C. I can see an air bubble in my Teriparatide-Teva Pen.	<p>A small air bubble will not affect your dose and it will not harm you. You can continue to take your dose as usual.</p>
D. I cannot get the needle off	<ol style="list-style-type: none">1) Put the large needle cover on the needle.2) Use the large needle cover to unscrew the needle.3) Unscrew the needle all the way by giving the large needle cover 3 to 5 counter-clockwise turns.4) If you still cannot get the needle off, ask someone to help you.
E. What should I do if I have difficulty pulling out the black injection button?	<p>Ask your doctor for advice, you may need to use a new Teriparatide-Teva Pen. When the black injection button cannot be pulled out, it means there is not enough medicine in your Teriparatide-Teva Pen for another dose. You may still see some medicine left in the cartridge.</p>

Cleaning and Storage

Cleaning Your Teriparatide-Teva Pen

- Wipe the outside of the Teriparatide-Teva Pen with a damp cloth.
- Do not place the Teriparatide-Teva Pen in water, or wash or clean it with any liquid.

Storing Your Teriparatide-Teva Pen

- Teriparatide Teva should be stored in a refrigerator (2 °C to 8 °C). You can use Teriparatide Teva for up to 28 days after the first injection, as long as the pen is stored in a refrigerator (2 °C to 8 °C).
- After each use, refrigerate the Teriparatide-Teva Pen right away. Read and follow the instructions in the *Package Insert* on how to store Teriparatide-Teva.
- Do not store the Teriparatide-Teva Pen with the needle attached. Doing this may cause air bubbles to form in the medicine cartridge.
- Store the Teriparatide-Teva Pen with the blue cap on.
- Do not freeze Teriparatide-Teva Pen. If the Teriparatide-Teva Pen has been frozen, throw the device away and use a new Teriparatide-Teva Pen.
- If the Teriparatide-Teva Pen has been left out of the refrigerator, do not throw the Pen away. Place the Pen back in the refrigerator and contact your doctor or pharmacist.

Unopened product

- The unopened product may be removed from the refrigerator and stored at a temperature up to 25 °C for one single period of up to 5 days, after which you should return it to the refrigerator (2 °C - 8 °C). Discard the unopened product if stored above 8 °C for more than 5 days.

Other Important Notes

- The Teriparatide-Teva Pen contains 28 days of medicine.
- Do not transfer Teriparatide-Teva to a syringe. This may result in you taking the wrong dose of medicine.
- Write down your first injection date on a calendar.
- Read and follow the instructions in the User Manual so that you use your Teriparatide-Teva Pen the right way.
- Check the Teriparatide-Teva Pen label to make sure you have the right medicine and that it has not expired.
- Do not use the Teriparatide-Teva Pen if it looks damaged. Look at the Teriparatide medicine in the cartridge. If the medicine is not clear and colourless, or if it has particles, do not use it. Contact your doctor or pharmacist, if you notice any of these.
- Use a new needle for each injection.
- During the injection, you may hear one or more clicks - this is normal.
- The Teriparatide-Teva Pen is not recommended for use by the blind or by those who have vision problems without help from a person trained in the proper use of the device.
- Keep your Teriparatide-Teva Pen and needles out of the reach of children.

Disposal of Pen Needles and Teriparatide-Teva Pen

- Before throwing away the Teriparatide-Teva Pen, be sure to remove the pen needle.
- Put used needles in a sharps container or a hard plastic container with a secure lid. Do not throw needles directly into your household garbage.
- The used pen may be discarded in your household garbage after you have removed the needle.
- Do not recycle the filled sharps container.
- Ask your healthcare provider about options to dispose of the sharps container properly.
- The directions regarding needle handling are not intended to replace local, healthcare provider or institutional policies.
- Dispose of the Teriparatide-Teva Pen 28 days after first use, even if it still has medication left in it.

1st use date ____/____/____
Throw away after ____/____/____

If you have questions or need help with your Teriparatide-Teva, contact your healthcare professional.

Manufacturer

Pliva Croatia Ltd,
Prilaz baruna Filipovica 25,
10000 Zagreb, Republic of Croatia.

This user manual was last revised in June 2024.